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- partial infection, poor bone quality
- ③ Patients with tooth grinding (bruxism), diabetes, hypertension
- ④ Patients with stress, pregnancy, and patients with risk of infection.

2) Caution

- ① Incorrect surgical treatment occurs damage of inserted fixture, and bone tissue around it. Our implant system must be operated by dentists and surgeons with enough training and experience in oral surgery, prosthetics as well as diagnosis and pre-operative planning.
- ② Clinician must use product after completely understanding all the instructions and precaution on product.
- ③ Product cannot be reused and used as intended purpose of each

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Manual for dental implant (Abutment)

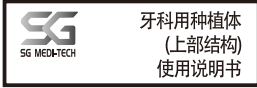
- 1. Performance & Intended use**
1) Performance
Abutments are dental implant components used for restoration of patient's chewing function and it is used as post to support prosthesis by connecting to fixture.
- 2) Purpose for use
When patient's tooth lost its chewing function, dental fixture & abutments system (artificial tooth) is used for restoration of the function. Especially, Abutment is connected to fixture and plays a role of post & cemented abutment to support prosthesis.
- 2. Method for use & operation**
1) Preparation before use
① This Product is non-sterile, so it has to be sterilized prior to use in patient's oral.

- ② Sterilization: this product has to be autoclaved for 5 minutes at 121°C (250°C).
- ③ Select the proper abutment fitting to specialty of each patient's case throughout carefully reading x-ray.
- ④ Inspect packing status, validation, damage of product.
- 2) Method for use
① Perform treatment when sufficient osseointegration occurs between bone and inserted fixture.
- ② Select adequate abutment by considering patient's oral condition.
- ③ Place the abutment into planned position without touching foreign material around.
- ④ Remove healing cap and healing abutment by driver, and connect selected abutment to fixture.

- ⑤ Taking impression by use of impression coping.
 - ⑥ Make a lab model by impressing obtained, and make prosthesis based on the model!
 - ⑦ Complete prosthesis by grinding and cleaning it.
 - ⑧ Mount prosthesis onto patient's oral by use of fixing screw & cement.
 - ⑨ Screw for abutment should be tightened with proper torque value (around 25 N.cm)
- 3. Precaution in use.**
1) Contraindication
① Patients with heart disease, hardening of arteries, coagulation, autoimmune disease, psychosis, tumor and patients taking steroid agents.
② Patients with poor oral hygiene,

4) This product is disposable. The re-use is prohibited.
5. Date: 17 Jan 2017 (Ver.05)

Manufacture
SHINSAEGI MEDI-TECH Co., Ltd.
#501, Gimhae BioMedical Center, 80-59, Goldenroot-ro, Juchon-Myeon, Gimhae-Si, Gyeongsangnam-do, Korea
Tel. 82-55-338-2815 Fax. 82-55-338-2818 E-mail : sgimplant@sgimplant.com

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